

User manual particle filtering half mask FFP3

Modell: Q-Pro XE



INTENDED USE

Q-Pro XS Particle filtering half mask is designed to protect against both solid and liquid aerosols.

These products are classed as Personal Protective Equipment (PPE) by the European PPE Regulation (EU) 2016/425 and have been shown to comply with this regulation through the Harmonized European Standard EN 149:2001+A1:2009 Filtering half mask.

Limitations:

- a. Not for use in atmospheres containing less than 79.5 percent oxygen,
- b. Not for use in atmospheres immediately dangerous to life or health;
- c. Do not exceed maximum use concentrations established by regulatory standards,
- d. Failure to properly use and maintain in this product could result in injury or death;

WARNING: that the user should not use the mask and enter the contaminated area if he cannot achieve a proper fit


PLEASE CHECK PRIOR TO USING TO

- a. The straps are attached well.
- b. The nosepiece and sponge are attached properly on the mask
- c. The position of valve is in proper location (valve model)
- d. The mask has not exceeded its shelf life: see date on the packaging
- e. Please do not use the mask if discovered any damages, holes. Dirt, contaminants on the mask

To protect your health and life, and to prevent serious injury or death, it is important to READ and FOLLOW the instructions enclosed with the product

HOW TO USE

- a. Hold the mask in hand with the metal nose piece up. Allow head harness to hang freely
- b. Position the mask under the chin, covering mouth and nose
- c. Pull the lower head harness over the head, position below ears. Pull the top head harness on the back of head above ears for best fit
- d. Hold the headband together and press on the adjustment clips
- e. Adjust tightness of head straps by moving adjustment clips
- f. Press soft metal nose-piece fit snugly around the nose
- g. To check, cup both hands over the mask and exhale vigorously. If air flows around nose, tighten the nose piece. If air leaks around the edge, reposition the head harness for better fit. Double check the seal and repeat the procedure until the mask is sealed properly

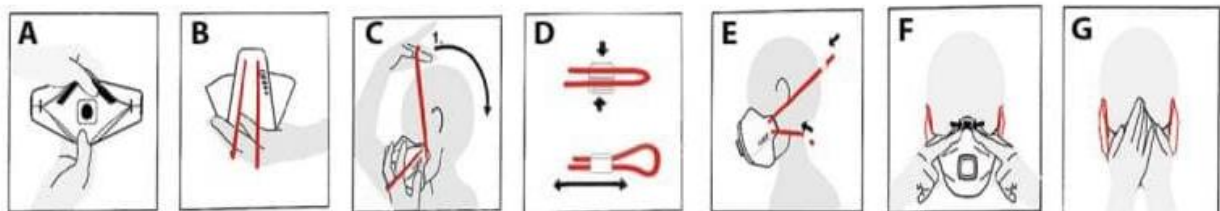

PLEASE READ ALL WARNINGS BEFORE

1. Failure to follow all instructions and limitations on the usage of this product could reduce the effectiveness of mask and may cause health injury or death

2. A proper selected mask is essential to protect your health. Before using this mask, please confirm with a safety consultant to make sure the mask is suitable for the conditions
3. This mask does not supply oxygen and must not be used in atmospheres containing less than 19.5% oxygen
4. Do not use when concentrations of contaminants may immediately cause danger to health or life
5. Get out of the working area and return to fresh air as soon as any difficulty to breathe, or dizziness or other distress occurs
6. To avoid leakage, do not use with beards or other facial hair that may interfere contact between the face and the edge of the mask
7. Never alter, modify, or abuse this mask
8. Single shift use only. Discard the mask right after single use
9. Replace mask immediately when it is hard to breathe or the mask is damaged or distorted, or a proper face fit cannot be maintained
10. Store unused masks in box in a clean, dry, non-contaminated area
11. Do not use in explosive atmosphere
12. This is to inform that the face mask shelf life is 3 years from the date of manufacturing if they are kept in their original package and stored under the following conditions: temperature between 5 and 38°C, and relative humidity of less than 75%

See information supplied by the manufacturer	Shelf life: 3 years	Temperature range of storage conditions 5-38°C	Maximum relative humidity of storage condition 75%
--	------------------------	--	--

WARNING: This respirator helps protect against certain contaminants. It may not eliminate the risk of contracting disease or infection. Misuse may result in sickness or death

PRODUCT NAME : Particle filtering half mask

MODEL : Q-Pro XE CLASS: FFP3 NR

SIZE : Folding size 21.5 x 7.6 cm (±0.5cm)

FFP3 NR defined in EN 149:2001+A1:2009

Compliant to EN Standards: EN 149:2001+A1:2009









PRODUCED BY:

UNIQON Protection AS
Hornelandsvegen 192, 5412 Stord
NORWAY

CERTIFIED BY:

0370 - LGAI TECHNOLOGICAL CENTER (ALLPLUS) C/ CAMPEZO Nº1 PARQUE EMPRESARIAL
LAS MERCEDES, 28022, MADRID, MADRID, SPAIN

SYMBOL ON THE LABEL

Symbol	Explanation	Symbol	Explanation
	Handle with care		Grønt Punkt Norge
	Keep dry		The mask sort as Rest
	Keep away from sunlight		The bag sort as plastic
	See information supplied by the manufacturer		The box sort as paper